



USER GUIDE FOR DASRI BAGS

These recommendations in no way replace the instructions issued within the establishment.

Recommendations prior to use

- Read the inscriptions on the bag
- Follow the precautions for use (do not use sharp objects to remove the strip holding the roll of bags)
- Identify the filling level
- Take care to use the bag with a suitable holder

Recommendations during use

- This bag should not contain anything other than soft infectious waste.
- The bag must not contain:
 - Sharp/cutting elements
 - Liquids
- Do not exceed the filling limit
- Close the bag using the implement provided

Recommendation for disposal, after use

- Do not reopen
- Do not place DASRI bags in other containers not subject to DASRI regulations or regulations on the transport of hazardous materials
- Place the bags in the dedicated collection area